





AVA DENT®

Digital Education Solutions

CLINICAL PROTOCOLS



CONSULTATION APPOINTMENT:

- a. Initial examination
- b. Diagnostic impressions (or take Putty impression of the intaglio of the existing denture)

Send Diagnostic impressions or Putty cast to Lab for custom trays and wax rims.

DOCTOR

- a. Fabricate custom trays and wax rims
- b. Send to doctor



LAB & GDS





FIRST TREATMENT APPOINTMENT: 45 MIN

- a. Final impressions using custom trays
- b. Final Wash in wax rims
- c. Adjust wax rims, OVD, midline, occlusal plane, incisal edge, tooth/shade selection
- d. OPTIONAL: Capture CR by Guided Closure or Gothic Arch Tracing
- e. Bite registration

Send records to lab for scanning

DOCTOR

LAB & GDS



- a. Scan records that have been sent from doctor
- b. Send scan files to GDS
- c. GDS designs denture setup and returns to lab
- d. Lab checks design and sends to doctor for approval





Using 3D Viewer software, doctor views denture setup and sends approval to the lab

DOCTOR

LAB & GDS

- a. Lab sends the approval to GDS
- b. GDS fabricates a try-in if requested





SECOND TREATMENT APPOINTMENT: 30 MIN

- a. Try-in of denture and make any necessary adjustments on esthetics and fit if necessary
- b. Do another wash in denture if required

SENT TRY-IN TO LAB WITH INSTRUCTIONS ON ANY CHANGES NEEDED

DOCTOR

LAB & GDS



- a. Lab receives try-in and does a re-scan if needed
- b. Lab sends try-in or new scan to GDS
- c. GDS fabricates the final denture and returns to lab, along with “Teeth Forever” card for patient
- d. Lab sends final denture and “Teeth Forever” card to doctor





THIRD TREATMENT APPOINTMENT: 20 MIN

- a. Deliver final denture
- b. Present "Teeth Forever" card to patient

DOCTOR

AVA DENT®
Digital Education Solutions